

# How to use your ELLIPTA-INHALER

Mouthpiece

Cover

ELLIPTA

29  
30

Dosage counter

## OPEN

Open the inhaler by sliding the cover down until you hear a **click**. The number on the dosage counter is now one unit lower, and your inhaler is ready for use.



## INHALE

Breathe out effortlessly for as long as possible. Then surround the mouthpiece with your lips – **do not cover the ventilation openings** – and breathe in long, deep and even through the inhaler. Hold your breath for at least 3 to 4 seconds and then breathe out. **Caution: do not breathe out into the inhaler.**

ELLIPTA

29

## CLOSE

Slide the cover back again and rinse your mouth out with water.

# How to use your ELLIPTA-INHALER

Mouthpiece

Cover

ELLIPTA

29  
30

Dosage counter

## OPEN

Open the inhaler by sliding the cover down until you hear a **click**. The number on the dosage counter is now one unit lower, and your inhaler is ready for use.



## INHALE

Breathe out effortlessly for as long as possible. Then surround the mouthpiece with your lips – **do not cover the ventilation openings** – and breathe in long, deep and even through the inhaler. Hold your breath for at least 3 to 4 seconds and then breathe out. **Caution: do not breathe out into the inhaler.**

ELLIPTA

29

## CLOSE

Slide the cover back again and rinse your mouth out with water.



## ELLIPTA APPLICATION VIDEO ON THE INTERNET

Enter “Ellipta Anwendungsvideo” (Ellipta application video) into your search engine and watch the demonstration of correct inhalation.



A.MENARINI PHARMA GMBH